

MiniCollect® Capillary Blood Collection System


...simple handling with improved safety

- ▶ Capillary made of specially coated plastic
- ▶ Cap with an automatic re-sealing capability
- ▶ Specially designed funnel made of Polypropylene
- ▶ Plastic tube with graduation marks
- ▶ Carrier tube for centrifugation
- ▶ Safety Lancet with various puncture depths
- ▶ "Cross Cuts" Cap for filling and collection of the blood sample without the need for opening the tube. Therefore preventing dangerous aerosols and possible contamination



Instructions for use:

Clean the puncture area. Remove the lancet protective cover by twisting the cover 45°.

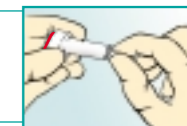

Puncture the heel area (neonates) by pressing in the coloured button to perform the puncture.



Puncture the fingertip (children). Hold the puncture finger between the thumb, pointer and middle fingers. Press in the coloured button to perform the puncture.



Using slight thumb pressure form a blood drop at the puncture site. With the aid of a capillary or funnel, transfer the blood into the tube. Capillary: Blood will flow effectively into the tube as a result of the capillary effect.



Funnel: Ensure that the funnel handle is positioned over one of the corners of the cap.



After blood collection, gently remove the funnel or capillary from the tube and dispose of. The cap closes automatically. The specimen should be adequately mixed by gently inverting the tube without removing the cap. A gentle tap to the bottom of the tube will ensure proper mixing and additive performance.



- Flexible rubber cap with "Cross Cuts" which makes de-capping of the tube unnecessary
- Maximum safety through a closed system
- Suitable for use with a funnel or capillary
- Graduation marks to determine exact filling amount
- A single tube for blood collection and centrifugation
- New improved tube design with free standing ability



MiniCollect® Capillary Blood Collection System

MiniCollect® Capillary Blood Collection System

Tubes

Order No.	Description	Cap colour	Fill volume	Area of Application
450400	Serum / Clot activator	red	1 ml	Serum determinations
450401	Serum Gel / Clot activator	red	0.8 ml	Serum determinations
450443	Serum / Clot activator	red	0.5 ml	Serum determinations
450444	Serum Gel / Clot activator	red	0.5 ml	Serum determinations
450444	Serum Gel / Clot activator	red	0.25 ml	Serum determinations
450403	EDTA K3	lavender	1 ml	Haematology
450404	EDTA K3	lavender	0.5 ml	Haematology
450410	EDTA K3	lavender	0.25 ml	Haematology
450405	Lithium Heparin	green	1 ml	Plasma determinations
450406	Lithium Heparin Gel	green	0.5 ml	Plasma determinations
450446	Lithium Heparin	green	0.5 ml	Plasma determinations
450407	Sodium fluoride / Potassium oxalate	grey	1 ml	Glucose
450445	Sodium fluoride / Potassium oxalate	grey	0.25 ml	Glucose
450409	Sodium citrate 3.8 %	blue	1 ml	Coagulation (only suitable for venous blood)
450413	Sodium citrate 3.2 %	blue	1 ml	Coagulation
450412	No additive	white		

Individually packaged as 100 pieces, total 1000 pieces

Accessories

Order No.	Description	Quantity
450419	Carrier tube (Bilirubin)	100 pcs. / total 1000 pcs.
450420	Carrier tube	100 pcs. / total 1000 pcs.
450421	Funnel	100 pcs. / total 1000 pcs.
450424	Lancet 1 mm, yellow	200 pcs. / total 1200 pcs.
450425	Lancet 1.5 mm, orange	200 pcs. / total 1200 pcs.
450426	Lancet 2.00 mm, red	200 pcs. / total 1200 pcs.
450431	Capillary (non-coated)	100 pcs. / total 1000 pcs.
450432	Capillary for Sodium fluoride	100 pcs. / total 1000 pcs.
450434	Capillary for EDTA	100 pcs. / total 1000 pcs.
450436	Capillary for Heparin	100 pcs. / total 1000 pcs.

Addresses of Greiner-Bio-One Subsidiaries

Greiner Bio-One GmbH
Bad Haller Strasse 32
A - 4550 Kremsmünster
Austria / Europe
Tel: (+43) 7583 6791 - 0
Fax: (+43) 7583 6318

St. Leonhardstrasse 31
9000 St. Gallen
Switzerland
Tel: (+41) 71 228 55 22
Fax: (+41) 71 228 55 21

Krablerstrasse 127
D - 45326 Essen
Deutschland
Tel: (+49) 201 861 860 - 0
Fax: (+49) 201 861 86 - 12

Po. Castellana 95
28046 Madrid
Spain
Tel: (+34) 91 555 60 39
Fax: (+34) 91 556 85 62

Stroudwater Business Park
Brunel Way
Stonehouse, Glos. GL10 3SX
United Kingdom
Tel: (+44) 14 53 82 52 55
Fax: (+44) 1453 82 72 77

Rua Clodomiro Franco de Andrade Jr. 125
Trevô Campinas SP, CEP 13035 - 481
Brazil
Tel : (+55) 1 927 2 8700
Fax : (+55) 1 927 25 666

Albert Einsteinweg 16
2408 AR Alphen a/d Rijn
The Netherlands
Tel: (+31) 172 42 09 00
Fax: (+31) 172 44 38 01

Fertősor 7
H - 9200 Mosonmagyaróvár
Hungary
Tel: (+36) 96 213 088
Fax: (+36) 96 213 198

4238 Capital Drive
Monroe, North Carolina 28112
U.S.A.
Tel: (+1) 704-261-7800
Fax: (+1) 704-261-7899
Customer service: 888-286-3883

6 Place Sainte Croix
F - 86000 Poitiers
France
Tel: (+33) 5 49 88 75 88
Fax: (+33) 5 49 88 76 01

Regional Office - Maghreb
16 Lebanon St.
Mohandeseen
Giza
Egypt
Tel: + 202 303 2571 / 303 2576
Fax: + 202 303 2524
Mobile: + 2012 218 9180
e-mail:

MiniCollect®
Capillary Blood Collection System
for infants and neonates



© Greiner 980029 Rev 04.04.2009 0

www.greiner-bio-one.com

MiniCollect®
by VACUETTE

greiner bio-one