



## MiniCollect® Safety Lancets

Gentle & Caring for Patients Both Young and Old



- Safe and hygienic capillary blood collection
- More precision for simple sample collection
- Unique identification of different types with colour code
- Practical handling for increased collection comfort

## More Safety



- Absolute safety due to the automatic retraction of the blade into the plastic housing after puncture
- The blade is only activated when the lancet has been positioned on the skin and the activation button is pressed
- Designed for single-use only

## Precision and Simple Handling




- Each version ensures a consistent puncture depth for straightforward sample collection
- The standardised blade positioning means reliably consistent puncture alignment
- Ergonomic design for more handling comfort
- Self-explanatory application saves time
- Small, compact product design saves space in disposal container

## Clear Identification

Available in three different puncture depths, which are identifiable with three different colours.

## Application

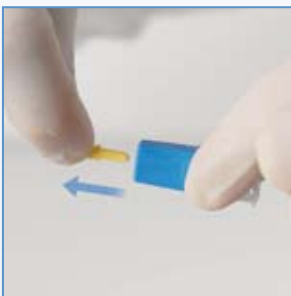

**1** Remove the protective cover



**2** Position the lancet



**3** Puncture the skin by pressing button



**4** Discard first drop of blood

## Ordering Details

### MiniCollect® Safety Lancets

Item no.	Description	Blood Volume	Inner pack	Outer pack
450427	MiniCollect® Safety Lancet, pink, penetration depth 1 mm	Low	200	2000
450428	MiniCollect® Safety Lancet, green, penetration depth 1.5 mm	Medium	200	2000
450429	MiniCollect® Safety Lancet, blue, penetration depth 2 mm	High	200	2000