


Product Information

VACUETTE® Speedy Quick Release Holder

General Information

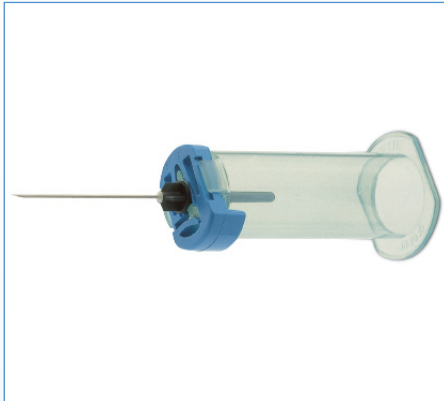

After venipuncture the needle is simply released from the holder by pushing the release mechanism so that the needle falls directly into a sharps disposal container.

The release mechanism is designed to retract automatically. It is recommended to use this device as a single-use holder.

Material

Halter: Polypropylen (PP)

Mechanismus: Polyoxymethylen (POM)

Method of cleaning

Where an inspection of the Speedy before or after use indicates visible contamination, proceed with the following:


For cleaning purposes, soak the holder in a disinfectant solution (e.g. isopropyl alcohol) recommended for medical instruments, in accordance to solution guidelines.

or:

Clean in a washing machine for medical instruments (80 °C).

In the case of the holder remaining damaged, discoloured, visibly contaminated or displaying signs of wear, dispose of it immediately.


Handling

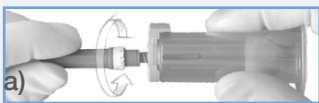

- Thread the needle (in a clockwise direction) into the holder. (a)
- Make sure that the needle is firmly seated in the holder.

NOTE: Do not over-twist the needle as this can lead to loosening of the needle during collection!

*Please ensure that the **needle position is completely straight**. If it is not straight, there is no guarantee that the needle remains firm in the holder.*

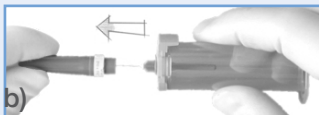

- Remove the protective sleeve from the needle. (b)

- Proceed with venipuncture.



- Hold the Speedy **vertically close above a sharps disposal container** with the needle tip pointing downwards. (c)
- Dispose of the needle by pushing in the extended part of the blue release mechanism.

Due to the spring feature in the release mechanism, the device automatically retracts and is ready for the next use. In some countries, where the usage of multiple-use holders is not prohibited by law, it can theoretically be used approximately up to 100 times.