**TRANSTUBE® with Stuart's Medium
MW163 & MW164**

Transtube® is a unique transport system for the recovery of aerobes and anaerobes incorporating Stuart's liquid medium. **Transtube®** offers significant advantages over contemporary liquid based transport systems particularly as the transport medium is retained in a soft foam pad at the base of each transport tube.

INSTRUCTIONS FOR USE

1. Peel back pouch until applicator and Transtube caps are exposed.
2. Remove Transtube cap between forefinger and thumb and discard.
3. Remove applicator and take sample.
4. Place applicator into Transtube and push down firmly until the Bell cap reaches line.
5. Squeeze bottom of tube to release media
6. Fill in patients details.
7. Transport to laboratory immediately.

FORMULA:

Sodium glycerophosphate
Sodium thioglycollate
Cysteine hydrochloride
Calcium chloride

pH 7.4 ± 0.2

Material safety information

Transtube plastic components do not contain latex or PVC.

FINAL PRODUCT QC

Samples are tested to ensure that the minimal inocula of a spectrum of organisms are recovered after a specified duration of time. Suspensions of the following organisms are absorbed on to the swab of each product sample and placed in its respective medium.

Aerobes & Facultative anaerobes

Pseudomonas aeruginosa
Streptococcus pyogenes
Streptococcus pneumoniae
Haemophilus Influenzae
Staphylococcus aureus
MRSA

Anaerobes

Bacteroides fragilis

Fastidious

Neisseria gonorrhoeae

MEDICAL WIRE & EQUIPMENT
CORSHAM, WILTSHIRE, ENGLAND SN139RT
Telephone (01225) 810361, Facsimile (01225) 810153
Email: info@mwe.co.uk Internet: www.mwe.co.uk
REVISION: JULY 09

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