



# PRECISE AND SAFE FOR SMALL AND FRAGILE VEINS



## CLiP® NEO

### Automatic safety IV catheter

CLiP® Neo is an automatic safety IV catheter designed for small veins found in the neonatal, paediatric or elderly patient groups. CLiP® Neo is available in 24 and 26 Gauge to cater to demanding physical conditions and different clinical needs. CLiP® Neo has been developed with the user and patient in mind and its use contributes to safe and successful insertion.

- / Instant blood response with optimized needle notch
- / Wing holder for multiple gripping alternatives
- / Lightweight to not affect handling
- / Stabilization wings for secure fixation to the skin
- / Blood sampling as indication for use
- / Small environmental footprint with compact product and packaging design

#### ONLINE TRAINING FOR HEALTH CARE PROFESSIONALS.

We offer online courses to support health care professionals with best practice when working with CLiP® Neo automatic safety IV catheter.

Register for your course at [stick-to-safety.com/training](https://www.stick-to-safety.com/training)



## PRODUCT OVERVIEW

FOR FURTHER INFORMATION AND/OR SAMPLE ORDERING  
PLEASE VISIT OUR WEBSITE OR CONTACT US.

### CLiP® Neo Automatic safety IV catheter

*Not made with natural rubber latex and DEHP and PVC free*

Item no.	Catheter material	Catheter size	Catheter dimension	Colour code*	Catheter flowrate	Qty. inner / outer
NW241901	FEP	24 G	0.7 x 19	● yellow	22	50 / 1,000
NW261901	FEP	26 G	0.6 x 19	● violet	15	50 / 1,000

FEP = Fluorinated Ethylene Propylene

\* Colour-coded according to ISO10555-5:2013

## RECOMMENDED HANDLING

Please observe the instructions for use at [www.stick-to-safety.com](http://www.stick-to-safety.com) or in the shelf box.



1

Insert the catheter and observe blood response between needle and catheter.



2

Occlude the vein and withdraw the needle straight back. Observe the safety clip activation.



3

Fixate and flush the catheter according to local protocol. Cover the site with a sterile dressing.