

FOR QUICK AND SAFE NEEDLE EJECTION



SPEEDY

Quick-Release Holder

The Speedy Quick Release Holder ensures safe and reliable ejection of the used blood collection needle into a sharps container, tested for use up to 100 times.

Simply by pressing the blue release mechanism on the holder ensures easy disposal of the needle in the sharps container – without having to touch the contaminated needle. Speedy is a safe alternative to conventional holders.

The mechanism features an arrow indicating which direction to push the release mechanism for ejection of needle.

- / No need to re-set the release mechanism before use
- / When reused: easy cleaning thanks to smooth surface
- / Up to 100 times proven usage

IMPORTANT TO KNOW.

The release mechanism features an optical orientation guide, which serves as an indicator for the correct positioning of the needle in the holder.

Furthermore the arrow form shows the direction for pushing the mechanism for ejection of the needle.



Disclaimer: Greiner Bio-One nevertheless expressly recommends single use.



Greiner Bio-One GmbH Kremsmünster, Austria
PHONE +43 7583 6791-0 / FAX +43 7583 6318 / E-MAIL office@gbo.com
Greiner Bio-One is a global player.
Find the contact details of your local partner on our website.

PRODUCT OVERVIEW

FOR FURTHER INFORMATION AND/OR SAMPLE ORDERING
PLEASE VISIT OUR WEBSITE OR CONTACT US.

Speedy Quick Release Holder

| Item No. | Inner / Outer [Qty.] |
|----------|----------------------|
| 450212 | 10 / 1.000 |

RECOMMENDED HANDLING

Please observe the instructions for use at www.gbo.com.



1

After successful blood collection, hold the Speedy Holder vertically with the needle tip pointing downwards.



2

Press the trigger mechanism.



3

Safely eject the needle into the container.