

PROVIDING HYGIENIC AND SAFE URINE COLLECTION FOR YOUR ROUTINE



IVD

URINE CUP with Integrated Transfer Device


The Urine Cup with Integrated Transfer Device allows you to collect and transport urine samples safely to the necessary destination. Thanks to the integrated transfer device, sample transfer from the cup to an evacuated tube is simple, fast and hygienic.

The optimized threading of the Urine Cup makes it easy to open and close. In addition, a stabilizing flange makes the cup closure more secure and prevents over-threading.

- / Recessed transfer device to reduce risk of needle stick injuries.
- / Available bulk packed or in cases of small packs to suit different distribution needs within healthcare facilities.

NEW LABEL DESIGN.

The perforated label acts as a sterility integrity seal visually indicating if the product has been opened prior to use.

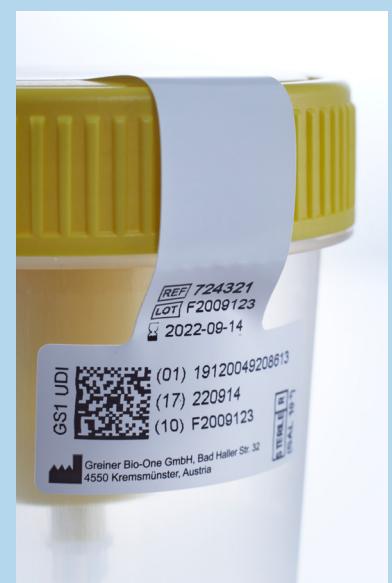

PRODUCT OVERVIEW

FOR FURTHER INFORMATION AND/OR SAMPLE ORDERING
PLEASE VISIT OUR WEBSITE OR CONTACT US.

Urine Cup with Integrated Transfer Device

| Item No. | Volume | Cap design | Lid Colour | Sterile | Qty. inner / outer |
|----------|--------|------------|------------|-------------|--------------------|
| 724321 | 100 ml | screw cap | ● yellow | sterile | - / 300 |
| 724322 | 100 ml | screw cap | ● yellow | non-sterile | - / 300 |
| 724325 | 100 ml | screw cap | ● yellow | sterile | 10 / 250 |

RECOMMENDED HANDLING

Please observe the instructions for use at www.gbo.com.



1

If more than one tube will be filled, loosen the lid slightly more than a quarter turn to normalize the pressure inside the cup.



2

Peel back the safety label.



3

Insert the tube and wait until it is completely filled. Remove the tube afterwards.



4

Place the safety label in its original position and re-tighten the lid.