19 STEPS OF VENOUS BLOOD COLLECTION

TRAINING AVAILABLE!

Visit **training.gbo.com** for more information

Greiner Bio-One recommends the following procedure



Physician's Instruction is received



Ensure all the required products are available




Hand hygiene



Patient identification and discussion



Position patient either in a sitting or lying position

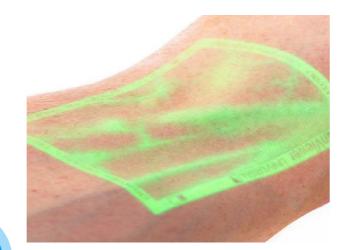

Select the puncture site



Hand hygiene, put on gloves



Product selection based on the patient



Disinfect the puncture site



Apply a tourniquet



Venepuncture



Fill the tube



Release the tourniquet



Take samples (correct order of draw) + invert the tubes



Withdraw and dispose of the needle



Label the tube



Cleaning and wound dressing



Special handling recommendations and transport



Farewell, disposal, cleaning and hand disinfection

WHO guidelines on drawing blood best practices in phlebotomy WHO (2010)

CLSI standards GP41 Collection of Diagnostic Venous Blood Specimens; 7th Edition. CLSI (2017)

Phlebotomy Essentials McCall R.; Tankersley C. M.; 6th Edition (2016)

Greiner Bio-One's expertise

NOTE

Always follow general effective and safe working procedures and precautions during use and disposal of the products.

Legal Disclaimer: The information contained in this document is the view of the authors and does not imply a formal endorsement on the part of Greiner Bio-One (or the outside organizations that employ the authors). It is provided for information purposes only and does not constitute medical advice nor is it intended or recommended as a substitute for professional medical advice. The information contained in this document is not a substitute for informed judgment and the reader shall be solely responsible for decisions that may arise in whole or in part from the use of or reliance upon information contained in the document. To the extent permitted by applicable law, neither Greiner Bio-One nor the authors shall be liable for any decision made, action taken or inaction in reliance on the information, statements, evaluations or recommendations set forth in this document.

www.gbo.com

Greiner Bio-One GmbH / Bad Haller Straße 32 / A-4550 Kremsmünster / Austria PHONE +43 7583 6791-0 / FAX +43 7583 6318 / E-MAIL office@gbo.com

