

HANDLING RECOMMENDATIONS

FOR THE
COMPLETE
HANDLING
INSTRUCTIONS

please refer to the
Instructions for Use
for each product on
www.gbo.com.

MINICOLLECT®

Capillary Blood Collection System



1 Prewarm and disinfect the puncture site.



2 Remove the cap.



3 Carry out the puncture and discard first drop of blood.



4 Collect the blood sample.



5 Close the tube with the original cap.



6 Immediately after collection, invert the tube gently.

ORDER OF DRAW

ACCORDING TO CLSI GP42-ED7

Note: Always follow your facility's protocol for order of draw.



1 EDTA

2 Other additive tubes

3 Tubes without additive and Serum tubes

MINICOLLECT® Coagulation Tube

Intended only for use
with venous blood.



1 Transfer venous blood into the tube.

2 Close the tube with the original cap.

3 Immediately invert the tube gently.

ADDITIONAL

WIDE RANGE OF SAFETY LANCETS

MINICOLLECT® PIXIE

Heel Incision Safety Lancets

light-green ●
↓ 0.65 mm

orange ●
↓ 0.85 mm

yellow ●
↓ 1 mm



MINICOLLECT®

Safety Lancets
with needle

lavender ●
28G / ↓ 1.25 mm

orange ●
23G / ↓ 2.25 mm

with blade

pink ●
↓ 1 mm

green ●
↓ 1.5 mm



blue ●
↓ 2 mm



MINICOLLECT®

Lancelino Safety Lancets
with needle

light green ●
30G / ↓ 1.2 mm

lavender ●
25G / ↓ 1.5 mm

blue ●
21G / ↓ 1.8 mm

green ●
21G / ↓ 2.4 mm

with blade

yellow ●
↓ 2 mm

The volume of
blood depends
on the length of
the lancet and
is marked with
blood drops



Legal Disclaimer: This product information is addressed exclusively to healthcare professionals. Devices of Greiner Bio-One are to be used by properly trained healthcare professionals only in accordance with the relevant Instructions for Use (IFU). For a listing of indications, contraindications, precautions and warnings, please refer to the Instructions for Use which accompanies each product or is available for download from our website at www.gbo.com (Download Center). For more information contact your local Greiner Bio-One sales representative or visit our website. All information is provided without guarantee despite careful processing. Any liability, warranty or guarantee of Greiner Bio-One GmbH is excluded. All rights, errors and changes are reserved. If not stated otherwise, Greiner Bio-One GmbH has all copyrights and/or other (user-)rights in this documents, in particular to signs such as the mentioned (word-picture-)brands and logos. Any use, duplication or any other use of the rights of Greiner Bio-One GmbH is expressly prohibited. Media owner: Greiner Bio-One GmbH

www.gbo.com

Greiner Bio-One GmbH / Bad Haller Straße 32 / A-4550 Kremsmünster / Austria
PHONE +43 7583 6791-0 / FAX +43 7583 6318 / E-MAIL office@gbo.com

Poster 980312 MiniCollect® Handling Recommendations en [rev.01, 07.2022]


greiner
BIO-ONE