



GENTLE CARE FOR YOUR YOUNGEST PATIENTS



MINICOLLECT® PIXIE Heel Incision Safety Lancet

MiniCollect® PIXIE is a safety lancet designed for capillary blood collection by heelstick. With penetration depths appropriate for use with newborn and premature babies, PIXIE provides the blood flow necessary for adequate collection for your testing purposes.

There are three MiniCollect® PIXIE safety lancet options available with varying incision lengths and penetration depths adapted for different stages of infant development.

The optimized ergonomic design and fully enclosed blade, with automatic retraction following activation, ensure your safety during capillary blood collection from the youngest patients.

- / Ergonomic design with non-slip, textured surface
- / Non-detachable safety tab to ensure no waste is left behind
- / One-handed activation for ease of use
- / Trigger orientation to minimize risk of tissue compression

PRECISE INCISION
WITH TAPERED BLADE
SLOT AND DUAL
COLOR.

When MiniCollect® PIXIE is positioned on the heel, the dual color design and arrow indicators on both sides allow precise location of the puncture site.



PRODUCT OVERVIEW

FOR FURTHER INFORMATION AND/OR SAMPLE ORDERS PLEASE CONTACT US OR VISIT OUR WEBSITE, WWW.GBO.COM.

MiniCollect® PIXIE Heel Incision Safety Lancet

sterile, with blade

Item No.	Color coding	Incision length	Penetration depth	Inner / Outer [Qty.]
450525	● light green	1.40 mm	0.65 mm	50 / 200
450526	● orange	1.75 mm	0.85 mm	50 / 200
450527	● yellow	2.50 mm	1 mm	50 / 200

RECOMMENDED HANDLING

Please observe the instructions for use at www.gbo.com.



1

Pre-warm and disinfect the puncture site according to your internal guidelines.



2

Press down on the safety tab to release the locking mechanism.



3

Position lancet, push trigger to activate and then dispose of PIXIE in a sharps container.



4

Apply gentle pressure to the surrounding tissue to form a blood droplet. Discard the first drop of blood (if appropriate) and begin collection with the next droplet.

NOTE: Please follow the recommendations of CLSI GP42, 7th Edition, on selection of the appropriate lancet for heel incision, taking into account the baby's weight and age.