



## MiniCollect® Lancelino Safety Lancets

Intuitive puncture makes all the difference



- ◀ Simple activation by applying pressure on the puncture site
- ◀ Ergonomically rounded design for comfortable grip
- ◀ Rounded base is a guide for pinpoint precision
- ◀ Exactly defined lancet lengths
- ◀ A choice of product versions to cover a range of applications and patients
- ◀ Automatic retraction of lancet after use

# Ease of Handling Ensures Product Safety



- Easy release: simply position on the puncture site and press
- Precisely defined needle and blade lengths
- The rounded design ensures a comfortable grip with no edges or ridges.
- The handling is simple and intuitive with little need for explanation.
- Safe, single-use product
- The various lengths of blade and needle lancets offer various puncture depths for use on younger and older patients.
- Lancets are colour-coded according to puncture depth for easy identification.
- The blade/needle is automatically retracted after puncture, thus preventing any risk of needlestick injuries to the user.

## Please observe the following Instructions for Use.

For the complete handling recommendations please refer to the Instructions for Use 980939 at [www.gbo.com](http://www.gbo.com).



**1** Remove the protective tab.



**2** Press the lancet against the puncture site until it activates.



**3** Disposal.



**4** Gently apply pressure to the surrounding tissue.

## Article Overview

Item n°	Description	Blood Volume	Packaging	
			Inner	Outer
<b>MiniCollect® Lancelino Safety Lancet with needle (sterile)</b>				
450520	Safety Lancet light green, 30G, penetration depth 1.2mm	●	200 pcs.	2000 pcs.
450521	Safety Lancet lavender, 25G, penetration depth 1.5mm	●●	200 pcs.	2000 pcs.
450522	Safety Lancet blue, 21G, penetration depth 1.8mm	●●●	200 pcs.	2000 pcs.
450523	Safety Lancet green, 21G, penetration depth 2.4mm	●●●●	200 pcs.	2000 pcs.
<b>MiniCollect® Lancelino Safety Lancet with blade (sterile)</b>				
450524	Safety Lancet yellow, 0.8mm, penetration depth 2mm	●●●●●	200 pcs.	2000 pcs.
<b>Product Sample Set and Give Away</b>				
980264	<b>MiniCollect®</b> Lancelino sample set (English)			
980262	<b>MiniCollect®</b> Lancelino plasterbox + children's tattoo (Give Away)			