



# GENTLE AND CARING FOR PATIENTS BOTH YOUNG AND OLD



## MINICOLLECT® LANCELINO Safety lancets

MiniCollect® Lancelino safety lancets are intended for single use only. Depending on the lancet type, blade or needle, different penetration depths are available.

MiniCollect® Lancelino safety lancets ensure enhanced safety when collecting capillary blood samples. Once the puncture has been performed, the blade/needle automatically retracts irreversibly into the plastic housing.





- / Simple activation by applying pressure on puncture site
- / Ergonomically rounded design for comfortable grip
- / Precise positioning thanks to round contact surface
- / Precisely defined penetration depths
- / A choice of product versions to cover a range of applications and patients

### WIDE RANGE OF LANCET TYPES

Both lancet type (needle/ blade) and insertion depth can be identified by the colour.

### BLOOD SAMPLE VOLUME

... when using a safety lancet with needle:

-  light green lancet
-  lavender lancet
-  blue lancet
-  green lancet

... when using a safety lancet with blade:

-  yellow lancet



## PRODUCT OVERVIEW

FOR FURTHER INFORMATION AND/OR SAMPLE ORDERING  
PLEASE VISIT OUR WEBSITE OR CONTACT US.

### MiniCollect® Lancelino safety lancets

sterile, for capillary blood collection

Item No.	Colour code	Blade	Needle	Blade/needle size	Penetration depth	Inner / Outer [Qty.]
450524	● yellow	X		0.8 mm	2 mm	200 / 2000
450520	● light-green		X	30 G	1.2 mm	200 / 2000
450521	● lavender		X	25 G	1.5 mm	200 / 2000
450522	● blue		X	21 G	1.8 mm	200 / 2000
450523	● green		X	21 G	2.4 mm	200 / 2000

## RECOMMENDED HANDLING

Please observe the instructions for use at [www.gbo.com](http://www.gbo.com).



1

Pre-warm and disinfect the puncture site according to your internal guidelines.



2

Pull off the protective cover.



3

Press on puncture site until lancet activates and then dispose of it.



4

Gently apply pressure to the surrounding tissue and discard the first drop of blood.