

Product Overview

VACUETTE® CLIX Safety Hypodermic Needle							
Item N°	Description	Gauge	Length	Indications	packaging		Availability
					inner	outer	
411250	VACUETTE® CLIX Safety Hypodermic Needle cream, sterile, latex-free	19G 1,1 mm	1 1/2" 38 mm	IM, IV	100 pcs.	800 pcs.	60 WD
411251	VACUETTE® CLIX Safety Hypodermic Needle cream, sterile, latex-free	19G 1,1 mm	1" 25 mm	IM, IV	100 pcs.	800 pcs.	60 WD
411252	VACUETTE® CLIX Safety Hypodermic Needle yellow, sterile, latex-free	20G 0,9 mm	1 1/2" 38 mm	BC, IM, IV	100 pcs.	800 pcs.	i.s.
411253	VACUETTE® CLIX Safety Hypodermic Needle green, sterile, latex-free	21G 0,8 mm	1 1/2" 38 mm	BC, IM, IV, IA	100 pcs.	800 pcs.	i.s.
411254	VACUETTE® CLIX Safety Hypodermic Needle green, sterile, latex-free	21G 0,8 mm	1" 25 mm	BC, IM, IV, IA	100 pcs.	800 pcs.	i.s.
411263	VACUETTE® CLIX Safety Hypodermic Needle green, sterile, latex-free	21G 0,8 mm	2" 50 mm	IM, IV, IA	100 pcs.	800 pcs.	60 WD
411256	VACUETTE® CLIX Safety Hypodermic Needle black, sterile, latex-free	22G 0,7 mm	1 1/2" 38 mm	BC, IM, IV, IA	100 pcs.	800 pcs.	i.s.
411255	VACUETTE® CLIX Safety Hypodermic Needle black, sterile, latex-free	22G 0,7 mm	1 1/4" 30 mm	BC, IM, IV, IA	100 pcs.	800 pcs.	60 WD
411259	VACUETTE® CLIX Safety Hypodermic Needle blue, sterile, latex-free	23G 0,6 mm	1" 25 mm	IM, IV, IA	100 pcs.	800 pcs.	i.s.
411257	VACUETTE® CLIX Safety Hypodermic Needle blue, sterile, latex-free	23G 0,6 mm	1 1/4" 30 mm	IM, IV, IA	100 pcs.	800 pcs.	60 WD
411262	VACUETTE® CLIX Safety Hypodermic Needle purple, sterile, latex-free	24G 0,55 mm	1" 25 mm	SC, IM, IA	100 pcs.	800 pcs.	60 WD
411258	VACUETTE® CLIX Safety Hypodermic Needle orange, sterile, latex-free	25G 0,5 mm	1" 25 mm	SC, ID, IM, IA	100 pcs.	800 pcs.	i.s.
411260	VACUETTE® CLIX Safety Hypodermic Needle orange, sterile, latex-free	25G 0,5 mm	1/2" 12,5 mm	SC, ID, IM, IA	100 pcs.	800 pcs.	60 WD
411261	VACUETTE® CLIX Safety Hypodermic Needle grey, sterile, latex-free	27G 0,4 mm	1/2" 12,5 mm	SC, ID	100 pcs.	800 pcs.	60 WD

BC - venous blood collection, SC - subcutaneous, ID - Intradermal, IM - intramuscular, IV - intravenous, IA -intraarterial

i.s. - in stock
60 WD - 60 working days lead time

Further items are available. They are available on request – 90 working days lead time apply. Minimum order quantity: 50.000 pcs.

A comprehensive overview list of all available product types can be sent upon request.



Austria (Main Office)
Greiner Bio-One GmbH
Phone +43 7583 6791-0
Fax +43 7583 6318
E-Mail office@at.gbo.com

For further information, please visit our website www.gbo.com/preanalytics or contact us.

Brazil
Greiner Bio-One Brasil
Phone +55 19 3468-9600
Fax +55 19 3468-9621
E-Mail office@br.gbo.com

Hungary
Greiner Bio-One Hungary Kft.
Phone +36 96 213 088
Fax +36 96 213 198
E-Mail office@hu.gbo.com

Netherlands
Greiner Bio-One B.V.
Phone +31 172 4209 00
Fax +31 172 4438 01
E-Mail info@nl.gbo.com

Thailand
Greiner Bio-One Thailand Ltd.
Phone +66 38 4656 33
Fax +66 38 4656 36
E-Mail office@th.gbo.com

China
Greiner Bio-One Suns Co., Ltd.
Phone +86 10 83 55 19 91
Fax +86 10 63 56 69 00
E-Mail office@cn.gbo.com

India
Greiner Bio-One INDIA Pvt., Ltd.
Phone +91 120 456 8787
Fax +91 120 456 8788
E-Mail info@gboindia.com

Portugal
VACUETTE Portugal S.A.
Phone +351 252 647 721
Fax +351 252 647 722
E-Mail info@vacuette.pt

Turkey
Greiner Bio-One Turkey
Phone +90 216 576 6004
Fax +90 216 576 6005
E-Mail office@tr.gbo.com

France
Greiner Bio-One SAS
Phone +33 1 69 86 25 25
Fax +33 1 69 86 25 35
E-Mail office@fr.gbo.com

Italy
Greiner Bio-One Italia S.r.l.
Phone +39 02 9438 3340
Fax +39 02 9438 3366
E-Mail office@it.gbo.com

Spain
VACUETTE Espana S.A.
Phone +34 91 652 33 35
Fax +34 91 652 33 35
E-Mail info@vacuette.es

United Kingdom
Greiner Bio-One Ltd.
Phone +44 1453 8252 55
Fax +44 1453 8262 66
E-Mail info@uk.gbo.com

Germany
Greiner Bio-One GmbH/Preanalytics
Phone +49 7022 948-0
Fax +49 7022 948-514
E-Mail office@de.gbo.com

Japan
Greiner Bio-One Co. Ltd.
Phone +81 3 3505 8875
Fax +81 3 3505 8945
E-Mail info@jp.gbo.com

Switzerland
Greiner Bio-One VACUETTE Schweiz GmbH
Phone +41 7 12 28 55 22
Fax +41 7 12 28 55 21
E-Mail office@ch.gbo.com

USA
Greiner Bio-One North America Inc.
Phone +1 704 261-7800
Fax +1 704 261-7899
E-Mail info@us.gbo.com



980985 rev.02, 06.2017 e



VACUETTE® CLIX Safety Hypodermic Needle

One simple “clix” to fix

VACUETTE® CLIX Safety Hypodermic Needle


One simple “clix” to fix

Greiner Bio-One aims to make the safest products available. By doing that, we contribute to the protection of healthcare workers from a risk of needle stick injuries.

This new product is therefore fitted with a safety shield and is manufactured in accordance with the European Biosafety Network recommendations for the implementation of the European Directive 2010/32/EU, which is compulsory for the European Community and represents best practice on a global level.¹

This device is to be used by properly trained healthcare professionals only in accordance with the relevant instructions for use.

The choice of two different safety activations methods - with thumb or on hard surface - makes the product a userfriendly one.

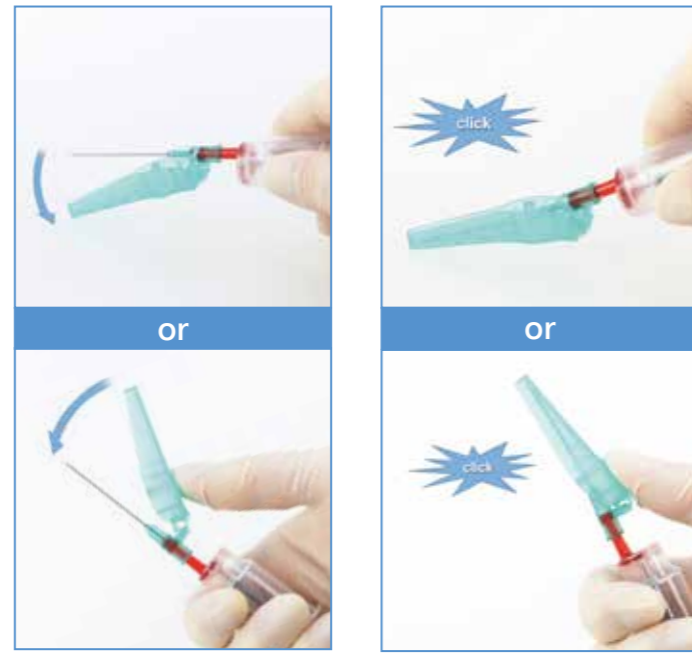


1 Activate safety mechanism on solid surface or with thumb

2 Audible click indicates safety mechanism activation

One product - two possible applications

The **VACUETTE® CLIX** Safety Hypodermic Needle can be used either for **venous blood collections**, or for **injections**.

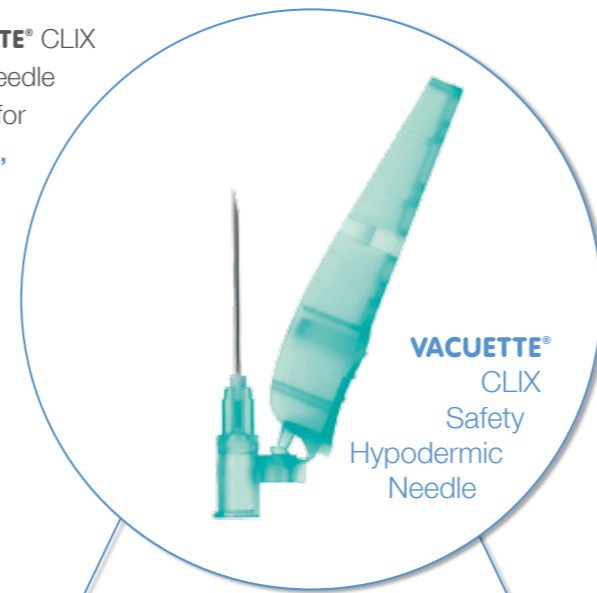

for venous blood collection

for assembly with holders e.g. HOLDEX® Single-Use Holder Item N°: 450263 and 450241

ALTERNATIVELY :
In cases where an evacuated blood collection is not possible: blood collection with a syringe, subsequent transfer into a tube by way of the Blood Transfer Unit (item N° 450225).



for injections

for use with commonly available syringes with a normed luer lock or luer slip

Products recommended for blood collection

- ◀ 20G, 1 1/2" Item N°: 411252
- ◀ 21G, 1 1/2" Item N°: 411253
- ◀ 21G, 1" Item N°: 411254
- ◀ 22G, 1 1/2" Item N°: 411256

Most frequently used needles for injection purposes

- ◀ 26G, 1/2" available on request
- ◀ 21G, 1 1/2" Item N°: 411253
- ◀ 23G, 1 1/4" Item N°: 411257
- ◀ 25G, 5/8" available on request


The perfect combination

VACUETTE® CLIX Safety Hypodermic Needle & HOLDEX® Single-Use Holder



Beneficial product features

- ◀ Intuitive handling, minimal training needed
- ◀ Choice of safety activation: with thumb, or on hard surface
- ◀ Extensive product line with further product types available
- ◀ Available in 8 lengths
- ◀ Single-packed, sterile
- ◀ Latex-free, non-pyrogenic, non-toxic
- ◀ Low-angle puncture when used in combination with a HOLDEX®

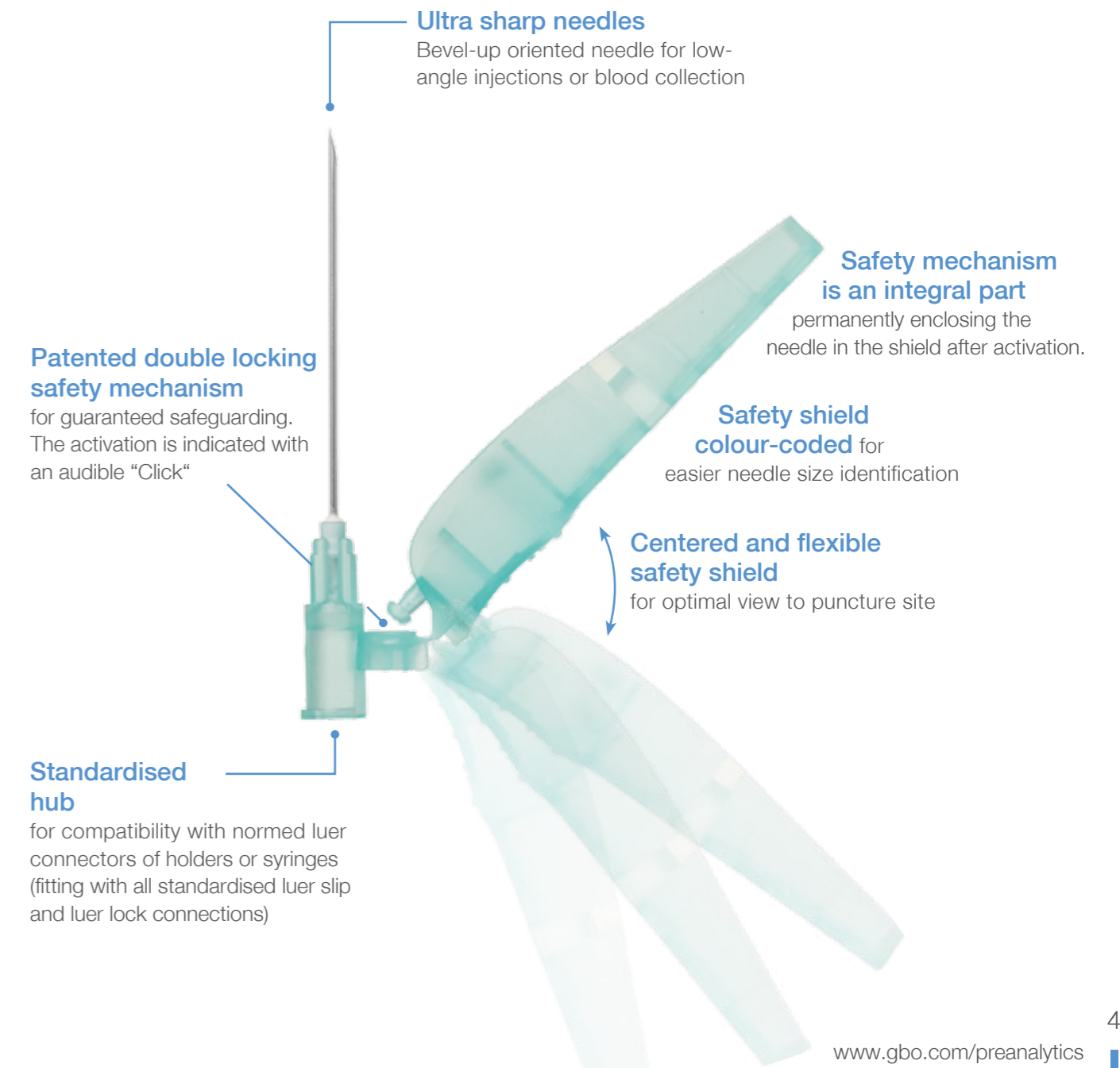

¹ Source: (European Biosafety Network: Implementation Guidance for the EU Framework Agreement, p.7, www.europeanbiosafetynetwork.eu