

**HOLDEX® Single-Use Holder PP**  
(sterile)

**Instructions for use**

Not available in the US!

Only applicable for the following item number: 450263

**Intended Use**

The HOLDEX® Single-Use Holder PP is for use in blood collection procedures in combination with blood collection devices with a female luer port.

**Product description**

A sterile, single-use, disposable plastic holder with an off-centre male luer connector at the top of the holder. A stainless steel needle with a sleeve is bonded to the inside of the holder to facilitate blood draw by puncturing the cap of an attached blood collection device.

**Handling**

1. Carefully open the packaging.
2. Grasp the holder in one hand and attach the male luer located at the top of the holder to the female luer of the chosen collection accessory. Make sure it is securely attached.
3. Carry out blood collection in accordance with your facility's policy.
4. After blood collection, dispose as described below.

**Storage**

Store holder at 4–25°C (40–77°F)

**NOTE:** Avoid exposure to direct sunlight. Exceeding the maximum recommended storage temperature may lead to impairment of the holder.










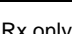
**Precautions/Cautions**

- The device will perform as intended when the instructions are followed accordingly.
- Always refer to the instructions for use for each individual accessory.
- Always wear gloves during a blood draw while using this device.
- Examine individual packaging for integrity prior to use. If the packaging has been broken, do not use.
- For single-use on a single patient only. Do not re-use because of the risk of harmful infections!
- Keep hands behind a needle at all times during use and disposal.
- Never touch the inside of the holder.
- Never recap a needle. The use of safety accessories is recommended.
- This device contains dry natural rubber and is correspondingly labelled on the packaging.
- Always check that tubes are filled correctly when using an evacuated blood collection system.
- The product is to be used by appropriately trained healthcare professionals only in accordance with these instructions.

**Disposal**

The general hygiene guidelines and legal regulations for the proper disposal of infectious material should be considered and followed. After use, dispose of the holder and all single-use accessories together in suitable biohazard disposal containers.

**Label information**

	Manufacturer		Temperature limit
	Use-by date		Do not re-use
	Batch code		Consult instructions for use
	Catalogue number		Contains or presence of natural rubber latex
	Sterilized using irradiation		Prescription Device



Greiner Bio-One GmbH  
Bad Haller Straße 32,  
4550 Kremsmünster  
Austria

[www.gbo.com/preanalytics](http://www.gbo.com/preanalytics)  
[office@at.gbo.com](mailto:office@at.gbo.com)  
Phone +43 7583 6791